

DIAGNOdent® pen 2190

Brief operating instruction

Select probe storage location:



MENU

Press twice



Press once

Choose storage space

Probe calibration with reference:




MENU


Press




Press
(within 3 Secs.)

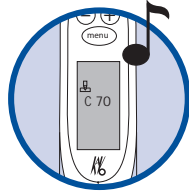

Watch the display until reference value shows and an audible signal is heard.

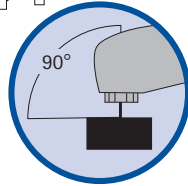


Hold probe onto reference ...

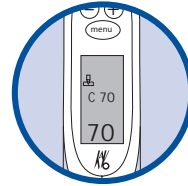


... until audible signal sounds. Calibration finished.

Customise according to patient:

Choose caries free spot - Press switch for 3 Seconds

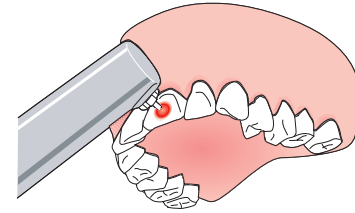

Occlusal-probe

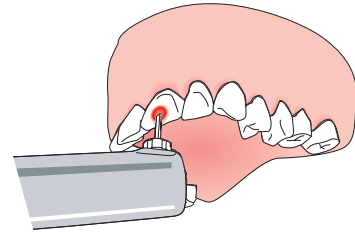

Approx-probe



KaVo. Dental Excellence.

KaVo Dental GmbH · D 88400 Biberach/Riß
 Telefon +49 7351 56-0 · Fax +49 7351 56-1488 · www.kavo.com



Those short operating instructions contain only the essential control functions. Before operating for the first time and in the case of operation by persons who are not familiar with handling this medical product, it is essential to read the Comprehensive Operating Instructions included in the delivery.

① Professional tooth cleaning

Removal of all hard and soft plaque with:

SONICflex,
 hand instruments,
 PROPHYflex with PROPHYpearls,
 Polishing handpiece DURAtec 2933

② Examination with DIAGNOdent pen 2190

